

GO IMPLANT-FREE

SeQuent® Please NEO

Safe and effective alternative to DES
in de novo lesions

ADVANTAGES OF DCB-ONLY


No unnecessary stent implantation

- No inflammation due to a foreign body implant
- No risk of stent thrombosis
- No stent-related limitations for further treatment
- No stent edge effect

Efficacy of DCB

- Enable positive remodeling
- Keep natural vessel vasomotion
- Only 1-month DAPT

DCB-only Treatment Methodology*

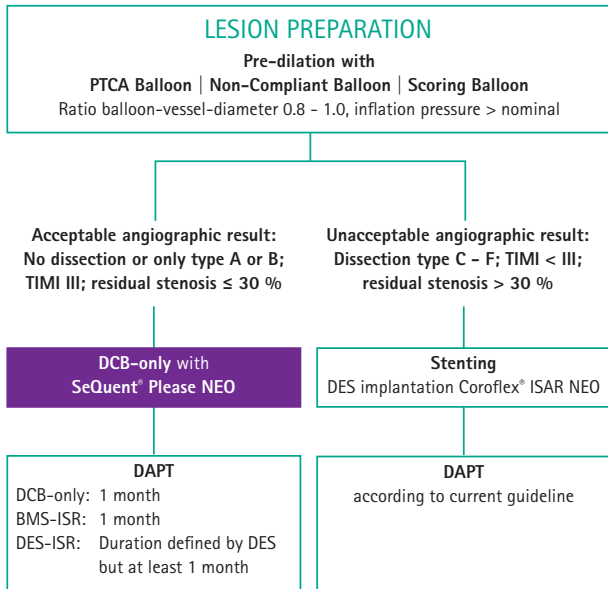

Important handling aspects SeQuent® Please NEO:

- Predilate the target lesion!
- Intended for local drug delivery
- DCB distal and proximal at least 2 - 3 mm longer than pre-dilated area
- Ratio balloon-vessel-diameter 0.8 - 1.0
- Inflation pressure 8 - 10 atm, time 30 seconds
- Do not bend, touch or wipe
- Do not expose any fluids to the catheter at balloon portion
- Position at the lesion site as fast as possible
- For single use only