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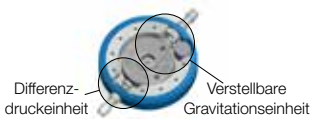
Patient	Selection of pressure levels		Opening pressure in total	
				
Newborns and children up to 3 years 	5*	20	5*	25
Children from 3 years 		25		30
Adults between 1,60 m and 1,80 m height 		25		30
smaller than 1,60 m		20		25
larger than 1,80 m		30		35
Adults from 65 years 		20		25
smaller than 1,60 m		15		20
larger than 1,80 m		25		30

All indicated pressure levels are in cmH<sub>2</sub>O. This is a non-binding recommendation for the attending physician. According to his diagnosis, the physician decides each case independently, without instructions and individually. The stated values consider the current scientific knowledge up to June 2018.

\* As special configurations in addition to the standard configuration the *M.blue* is also available with differential pressure units with pressure levels 0, 10, 15 cmH<sub>2</sub>O. In case of *M.blue plus* the opening pressure of the differential pressure unit (*proGAV 2.0*) is adjustable from 0-20 cmH<sub>2</sub>O.

Manufacturer:

 **CHRISTOPH MIETHKE GMBH & CO. KG**

Ulanenweg 2 | 14469 Potsdam | Germany  
Tel. +49 331 62 083-0 | Fax +49 331 62 083-40  
www.miethke.com

Distributor:

 **B. BRAUN**  
SHARING EXPERTISE

AESCULAP® – a B. Braun brand  
Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany  
Tel. 07461 95-0 | Fax 07461 95-26 00 | www.aesculap.de

## Standard pressure levels for *M.blue*® and *M.blue plus*®

The *M.blue* consists of a differential pressure unit and an adjustable gravitational unit.

The opening pressure of *M.blue* in the horizontal position is **only** determined by the differential pressure unit.

In the lying position the gravitational unit is not active.

The opening pressure in the vertical position is **the sum** of both: differential pressure unit and adjustable gravitational unit.

The M. Blue App demonstrates the functionality of *M.blue* and the effect of patient posture on valve opening pressure.

Now available in the App and Google Play Store.

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Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Deutschland