



Askina® Foam  
Askina® DresSil

**MORE TIME FOR CARING**  
EXUDATE MANAGEMENT

# Askina® Foam | Askina® Trachea

## Hydrophilic foam dressings

---

### INDICATIONS

---

**Askina® Foam** is indicated for the management of moderate exuding wounds, partial to full thickness wounds

- Pressure ulcers
- 1<sup>st</sup> and 2<sup>nd</sup> degree burns
- Venous and arterial leg ulcers
- Diabetic foot ulcers



### ADVANTAGES

---

- Fluid handling characteristics
- Maintain a moist environment



**Askina® Foam** is a two layered non-adherent foam dressing consisting of a soft hydrophilic polyurethane breathable foam layer and a thin, transparent, protective polyurethane film.

- Fluid handling capacity, waterproof and bacteria resistant
- Prevention of skin breakdown
- In case of venous leg ulcers, may be used under compression therapy<sup>(1)</sup>

**Askina® Trachea** is a sterile, hydrophilic foam dressing that includes a polyurethane foam wound contact surface with the circular aperture designed to fit neatly around a tracheostomy tube or other drain or stoma.

- Absorbent apertured foam dressing for the stoma created by tracheostomy or for wound drainage sites.

# Askina® Heel

Hydrophilic foam dressings

Anatomically shaped hydrocellular heel dressing



## ADVANTAGES

- Ready to use concept
- Innovative anatomical design covering the malleolus
- Pressure reducing foam, helps prevent stage I ulcers <sup>(1)</sup>
- Good protection against friction and shear stresses

**Askina® Heel** is an anatomically shaped, two layered, non adherent foam dressing, which includes a securement foam strap and two self adhesive hooks that allow dressing to remain in place even during movement.

- Stage I – IV pressure ulcers
- Surgical incisions
- 1<sup>st</sup> and 2<sup>nd</sup> degree burns



## HOW TO APPLY

- 1 Apply self adhesive labels on each side of **Askina® Heel**
- 2 Apply the dressing gently on the heel area and fix with the strap
- 3 Correctly positioned **Askina® Heel**: the ankle is covered
- 4 The dressing can be also fixed with a compression stocking

# Askina® DresSil

## Foam dressings with silicone adhesive

### INDICATIONS <sup>(1)</sup>

Askina® Dressil, Askina® Dressil Border, Askina® Dressil Heel, and Askina® Dressil Sacrum are indicated for the management of moderately exuding, partial to full thickness wounds such as:

- Pressure ulcers
- Venous leg ulcers
- Arterial leg ulcers
- Diabetic foot ulcers
- 1<sup>st</sup> and 2<sup>nd</sup> degree burns
- Donor sites
- Skin tears



Askina® DresSil is a self adherent foam dressing with soft silicone adhesive on one side and a vapour permeable waterproof film on the other. It combines the absorption capacity of the foam with the soft adhesion of the silicone contact layer.


### ADVANTAGES

The silicone adhesive layer provides several advantages:

- Gentle and secure adherence
- Minimized trauma during dressing changes <sup>(1)</sup>
- Good adaptability on difficult-to-dress areas and skin contours
- Can be cut to shape
- Improves patient comfort <sup>(2)</sup>

### STRUCTURE OF THE DRESSING

- 1 Protective polyurethane film • Highly breathable
- 2 Hydrophilic foam layer • Soft and conformable
- 3 Silicone adhesive layer • Safe adhesion to the surrounding skin for an atraumatic removal

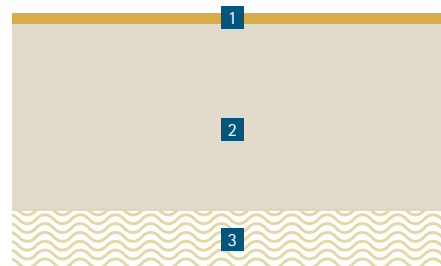

### EXAMPLE OF USE

Askina® DresSil may be used in conjunction with compression therapy <sup>(3)</sup>

1. Thomas S Atraumatic dressings. WorldWideWounds, 2003

2. Meuleneire F, Rücknagel H Soft Silicone Made Easy, Wounds International, May 2013

3. Instruction for use: Askina® Dressil, Askina® Dressil Border, Askina® Dressil Heel, Askina® Dressil Sacrum

# Askina® DresSil Border

Foam dressings with silicone adhesive

Additional adhesive border for extra security

---



## ADVANTAGES

---

Specially adapted for difficult-to-dress or moving areas (knees, elbows, skin folds)

Askina® DresSil Border has the same structure as Askina® DresSil, with an additional 1.5 cm large adhesive border.



## ADVANTAGES

---

- Specifically designed for diabetic foot ulcer
- Shaped island foam
- Suitable for small, deep wounds (diabetic foot ulcers)



## Special note

---

- Askina® DresSil Border allows the patient to take a shower

# Askina® DresSil Sacrum

## Foam dressings with silicone adhesive

---



### ADVANTAGES

---

- Gentle and secure adherence
- Minimized trauma during dressing changes <sup>(1)</sup>
- Good adaptability on difficult-to-dress areas and skin contours
- Improves patient comfort
- Pressure ulcers
- Skin tears

Askina® DresSil Sacrum coated with silicone all over the foam to allow intimate contact.

### SPECIAL NOTE

---

- Dressing changes will depend entirely upon the state of the wound and the amount of exudate



# Askina® DresSil Heel

Anatomically shaped foam dressing with silicone adhesive

---



## ADVANTAGES

---

- Atraumatic dressing removal <sup>(1)</sup>
- Good absorption capacity
- Soft and conformable
- Self adherent no need for secondary fixation
- Showerproof

Askina® DresSil heel can also be used for preventing pressure ulcers

- Protects the heel and the malleolus



## HOW TO POSITION

---

- 1 Remove the transparent top liner that is protecting the two "ears"
- 2 Position the foot on the dressing
- 3 Cover the ankle with the "ears" of the dressing  
Remove the transparent lower liner
- 4 Wrap and cover the heel with the lower part of the dressing

# Ordering information

Askina® Foam	Size	Pcs/Pack	Reference
	5 x 7 cm	10	7240710
	10 x 10 cm	10	7241010
	10 x 20 cm	10	7241210
	20 x 20 cm	5	7242005
Askina® Trachea			
	9,5 x 8,5 cm	10	7248510
Askina® Heel	Size	Pcs/Pack	Reference
	surface = 225 cm <sup>2</sup>	5	7240105

Askina® DresSil	Size	Pcs/Pack	Reference
	5 x 7 cm	10	5295710
	10 x 10 cm	10	5291010
	15 x 15 cm	10	5291510
	10 x 20 cm	10	5291210
	20 x 20 cm	5	5292005

Askina® DresSil Border	Size overall dressing   foam island	Pcs/Pack	Reference
	6 x 6 cm   3 cm Ø	10	5396610
	7,5 x 7,5 cm   4,5 x 4,5 cm	10	5397510
	10 x 10 cm   7 x 7 cm	10	5391010
	10 x 20 cm   7 x 17 cm	10	5391210
	15 x 15 cm   12 x 12 cm	10	5391510
	15 x 20 cm   12 x 17 cm	10	5395210
	20 x 20 cm   17 x 17 cm	5	5392005

Askina® DresSil Sacrum	Size overall dressing   foam island	Pcs/Pack	Reference
	16 x 17,5 cm   13 x 14,5 cm	5	5491605
	21 x 22 cm   17 x 18 cm	5   10	5492105   5492110

Askina® DresSil Heel	Size overall dressing   foam island	Pcs/Pack	Reference
	22 x 21,6 cm   18,9 x 18,5 cm	5	5592205

Manufactured by:

B. Braun Hospicare Ltd. | Collooney | Co. Sligo | Ireland

[www.bbraun.com/wound-management](http://www.bbraun.com/wound-management)

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and/or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun. Non-binding documents and photographs. For healthcare professional use only.