Askina® Scar Repair

en	Soft Silicone Dressing for Scar Management
de	Weiche Silikon-Auflage zur Narbenbehandlung
es	Apósito de silicona suave para el tratamiento de cicatrices
fr	Pansement silicone souple pour le traitement des cicatrices
it	Medicazione al silicone per il trattamento delle cicatrici
nl	Zacht siliconen verband voor litteken behandeling
bg	Мека силиконова превръзка за обработване на рани
cs	Měkké silikonové krytí pro hojení jizev
el	Μαλακό Επίθεμα Σιλικόνης για Διαχείριση Ουλών
et	Silikoonside armide raviks
fi	Pehmeä silikonisidosis arpien hoitoon
hu	Lágy szilikon kötszer sebhegek kezelésére
It	Minkšto silikonno tvarstis, skirtas randų priežiūrai
lv	Mīksts silikonu pārsejis rētu ārstēšanai
no	Myk silikonbandasje til arrbehandling
pl	Miękki opatrunek silikonowy do leczenia blizn
pt	Penso de silicone suave para a gestão da cicatriz
ro	Pansament moale din silicon, pentru tratamentul cicatricilor
ru	Появка с мягким силиконом для лечения рубцов
sk	Mäkké silikónové krytie na hojenie jaziev
sv	Mjukt silikonförfband för ärrbehandling
tr	Yara İzi Yönetimi İçin Yumuşak Silikon Örtü

en	INSTRUCTIONS FOR USE
1. PRODUCT DESCRIPTION	
Thin, conformable, reusable non sterile adherent dressing consisting of: <ul style="list-style-type: none">a vapour permeable, waterproof outer film and non woven laminate a soft silicone skin contact layer which conforms to the wound surface and adheres safely to the skin. <p>Askina® Scar Repair provides a degree of UV protection, blocking at least 30 % of UV-A and 60 % UV-B radiation</p>	
2. INDICATIONS	
Askina® Scar Repair is indicated for the management of hypertrophic and keloid scars and can also be used on closed wounds for prevention of hypertrophic or keloid scarring after surgery.	
It is indicated for external use only.	
3. PRECAUTIONS	
Do not stretch the dressing during application as tension can cause skin trauma and may cause the dressing to lift at the corners.	
To prevent skin irritation and to ensure good adhesion, make sure the skin is clean, free of soap residue and lotion and allowed to dry thoroughly before applying the dressing.	
Do not use if the pack is damaged or opened on initial use.	
4. CONTRAINDICATIONS	
If signs of irritation or infection develop or if the scar deteriorates unexpectedly, contact your professional authority.	
Caution if known allergy to silicone exists.	
The product cannot be used on open or infected wounds.	
5. INFORMATION FOR USE	
5.1 Skin Preparation	
Cleanse and dry the scar and area surrounding the scar.	
5.2 Dressing Application	
1) Select the appropriate size to cover the surface of the scar ensuring a minimum of 1cm margin beyond the scar edge. When using Askina® Scar Repair in conjunction with creams and lotions ensure that the dressing extends beyond the cream to avoid lack of adhesion.	
If a scar requires more than one dressing, these should be placed edge to edge instead of overlapping.	
2) Open the pouch and remove the dressing.	
3) The dressing can be cut to size if needed.	
4) Remove the backing liner and apply the adhesive side to the scar by gently smoothing the dressing.	
5) If the scar requires UV protection, take into account that Askina® Scar Repair blocks at least 30 % of UV-A and a 60 % UV-B radiation. If sun protection is complemented with sunscreens, consider point 1 of 5.2.	
5.3 Dressing Removal/Changes	
Askina® Scar Repair should be worn for 24 hours a day and removed once a day when showering or bathing to inspect and wash the skin. The dressing itself does not need to be washed.	
Under normal conditions the same Askina® Scar Repair can be used for 3 to 7 days or until the adhesive properties are no longer sufficient to keep the dressing in place.	
For prevention treatment, Askina® Scar Repair should be used for two to six months, depending on the condition of the scar.	
Askina® Scar Repair is waterproof and can be worn while bathing and showering.	
Askina® Scar Repair is easily removed without causing trauma.	
6. SPECIAL NOTES	
Topical silicone has been shown to have a positive impact on hypertrophic scars and keloids.	
It may take from three months up to a year or more to improve an old scar, depending on the condition of the scar tissue.	
Askina® Scar Repair provides ultra violet protection which is a medical recommendation for scar care.	
7. PRESENTATION	
Soft Silicone sheet dressing	
Soft Silicone strips dressing	

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