## Askin® Barrier Cream

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## EN INSTRUCTIONS FOR USE

1. **PRODUCT DESCRIPTION**  
Askin® Barrier Cream is a white concentrated cream which, when applied to the skin forms a protective layer and moisturizes dry skin.

2. **INDICATIONS**  
Askin® Barrier Cream is indicated for use on intact skin. Askin® Barrier Cream acts as a protectant and moisture barrier against maceration caused by incontinence or body fluids.  
• to protect sensitive and fragile skin, and severely dry skin, including peri-wound areas  
• to prevent skin irritation

## 3. CONTRAINDICATIONS / SAFETY INFORMATION

3.1 Askin® Barrier Cream should not be used on infected skin. If the skin condition worsens or does not improve within 7 days, contact the proper medical authority.  
3.2 Avoid contact with the eyes.  
3.3 For external use only; seek medical help in case of ingestion.  
3.4 Do not use if pack is damaged or opened.  
3.5 Use of other barrier products, creams, ointments or lotions may reduce the effectiveness of Askin® Barrier Cream.  
3.6 Askin® Barrier Cream should not be used where there is a known allergic sensitivity to the ingredients.  
3.7 Askin® Barrier Cream may increase the adherence of some adhesive products.

## 4. INGREDIENTS

Purified Water, Ethylhexyl Isononanoate, Hexamethyl Disiloxane, Polyisobutylene, VP Eicosene Copolymer, Hydroxyethyl Acrylate / Sodium Acryloyl Dimethyl Taurate Copolymer, Butylene Glycol, Oleth-3 Phosphate, Phenoxethanol, Hydrogenated Styrene/Isoprene Copolymer, Allantoin, Carbomer, Triethanolamine, Methylparaben, Propylparaben, Disodium EDTA.

## 5. INFORMATION FOR USE

Askin® Barrier Cream is very simple to apply, and requires no special skills or equipment. Re-application time will be dependent on the frequency of cleaning and with secondary dressing change. On frequently cleaned skin daily reapplication may be required, but this may be reduced to every 2 to 3 days for less frequent cleaning.

Cleanse the skin as required. Dry the skin thoroughly prior to Askin® Barrier Cream application. Spread Askin® Barrier Cream thinly to cover the entire affected area.

If Askin® Barrier Cream is applied to an area with skin folds or skin-to-skin contact, gently smooth the folds to apply the film, allow the cream to dry before letting the skin relax to its normal position.

If the feel is oily, too much has been applied. Remove excess cream with wipes.

Askin® Barrier Cream can be removed by using moist medical adhesive removers as directed.

## 6. PRESENTATIONS

Askin® Barrier Cream is available in the following size:  
• 28g Tube  
• 92g Tube

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