

Omnican® Lance soft

LANCETS FOR CAPILLARY BLOOD COLLECTION



For people with diabetes collecting blood for blood glucose monitoring is a big part of their lives. To make this as comfortable as possible B. Braun offers Omnican® Lance soft: With its thin and precise tip, the lancet ensures an adequate blood flow at minimized pain level. It makes the blood collection with a lancing device more comfortable and supports a successful diabetes therapy.

HIGH COMFORT

Lancet is available in two thin lancet tip versions so that you can choose the needle size that suits you most.

VARIED COMPATIBILITY

Rib design with standard dimensions provides compatibility with numerous lancing devices.


GENTLE PUNCTURE AND IDEAL BLOOD FLOW

A three-facets tip coated with silicone oil means less penetration force for excellent blood flow.









STERILITY

Lancet remains sterile as long as protective cap has not been removed.

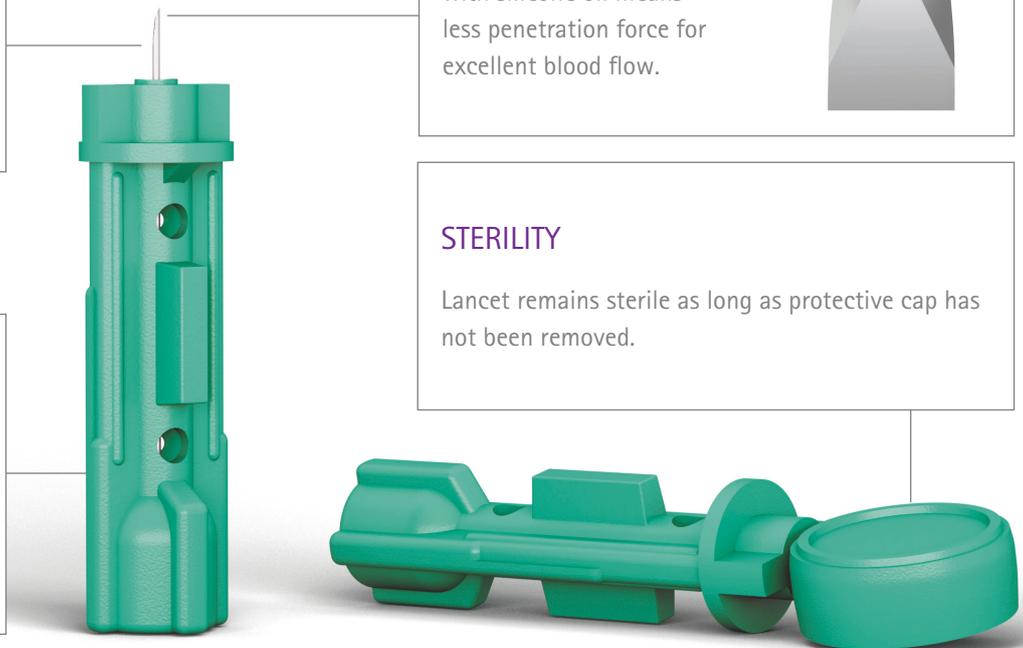

Omnican® Lance soft

HANDLING AT A GLANCE

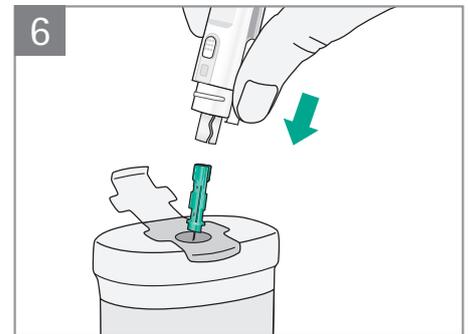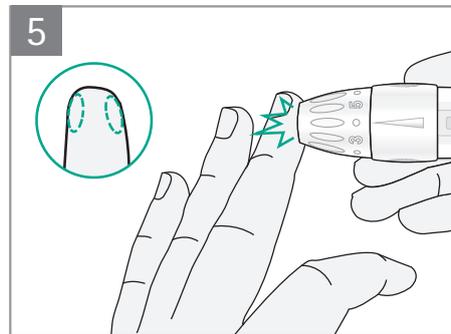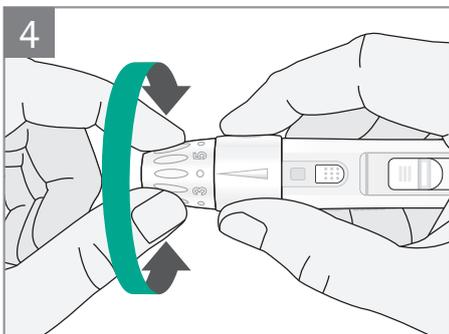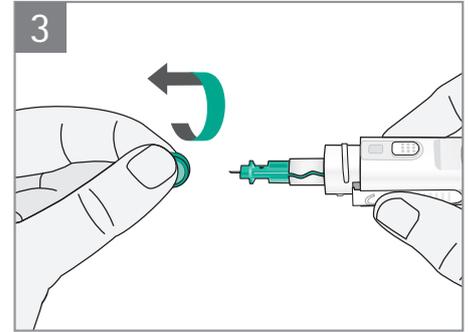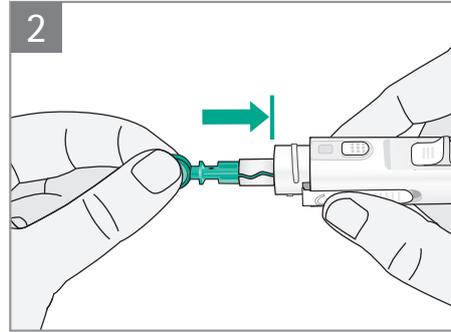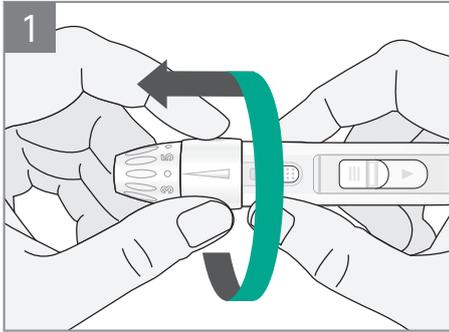

TIPS AND TRICKS

- Use a new sterile lancet for every blood sample
- Change puncture site systematically using the side of your fingertip
- Avoid index finger and thumb
- Always dispose of the used lancet properly in a suitable container

**LANCETS REMAIN STERILE
5 YEARS IN ACCORDANCE WITH
ISO 11137:1995**



VERSATILE COMPATIBILITY

B. Braun	Omnilance, Omnican Lance
Abbott	EasyTouch
Bayer	Microlet 2
LifeScan	One Touch UltraSoft
Menarini	Glucoject Dual Plus
Owen Mumford	Autolet Impression
Sanofi Aventis	MyStar
Ypsomed	mylife AutoLance, mylife Softlance

and other universal lancing devices.
The brands listed are the property of the named companies.

PRODUCT OVERVIEW

Omnican® Lance soft	REF
28G 200 pcs/pack	9167536E
30G 200 pcs/pack	9167510E
30G 50 pcs/pack	9167554A
Lancing device	REF
Omnilance®	9167609
Omnican® Lance	9167927_