



## FOCUS ON SAFETY

Solofix® Safety – effective protection against  
needlestick injuries in capillary blood collection

# SO SIMPLE. SO SAFE.

## Solofix® Safety – sterile single use safety lancets for capillary blood collection

Solofix® Safety single use lancet has been designed for modern capillary blood sampling. Solofix® Safety is an integral component of infection control programs in hospitals, clinics, laboratories, doctor's offices and wherever both patients and health care professionals need effective protection against infections through sharps injuries.

Solofix® Safety lancets are available in 3 different color-coded versions to obtain specific blood sample sizes and to comply with distinct skin types.

### EASY HANDLING

- Pre-activated safety mechanism
- Automatic irreversible needle retraction
- Defined blood flow through technique-independent pressure activation
- No risk of unintended activation

### HIGH PATIENT COMFORT

- Low trigger force activates double retraction spring system with fast puncturing mechanism
- Sharp needles (25 and 21G) with tri-bevel tip and precise linear guidance of the lancet minimize tissue damage and pain
- Ultra-sharp blade in Solofix® Safety Neonat ensures adequate blood volume and quick healing

### PREVENTING NEEDLESTICK INJURIES

- Safely protected needle before and after use
- Automatically inactivated system prevents reuse
- Complies with international safety guidelines



### Solofix® Safety Fine

Low blood flow –  
for blood glucose monitoring


Size	25G
Puncture depth	1.5 mm
Blood sample	< 10 µl
Sales unit (pcs/pack)	200
REF	6183010

25G

1.5 mm

### Instructions for use

- 1 Twist off and pull out protective front cap
- 2 To activate, press against puncture site
- 3 Obtain blood droplet as usual

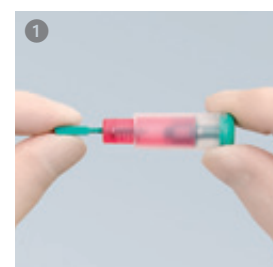

# ection



## Solofix® Safety Universal



Medium blood flow –  
for general blood diagnosis

## Solofix® Safety Neonat



High blood flow –  
for heel sticks in neonate screening  
and finger sticks in adults

Size	21G
Puncture depth	1.8 mm
Blood sample	10 - 30 µl
Sales unit (pcs/pack)	200
REF	6183000




21G

1.8 mm

Size/blade	0.8 mm
Puncture depth	2.0 mm
Blood sample	70 - 100 µl
Sales unit (pcs/pack)	200
REF	6183020

0.8 mm

2.0 mm

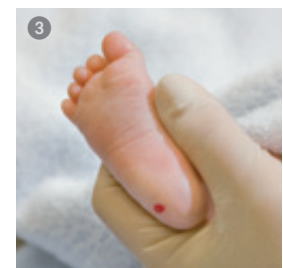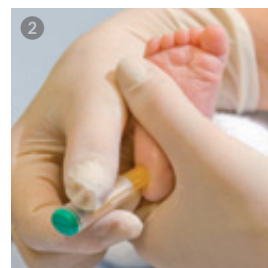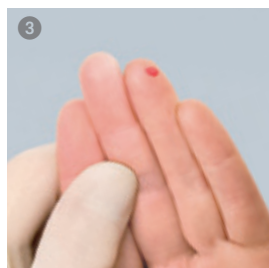

# CAPILLARY BLOOD COLLECTION



## Solofix®

- Standard steel lancet for capillary blood collection
- Sterile single-blistered
- Stainless steel, nickel-free
- Asymmetric point

Size	Sales unit (pcs/pack)	REF
2.9 mm exposed length	200	6182003
2.9 mm exposed length	500	6185002



## Omnican® Lance soft

- Sterile disposable lancets for gentle puncture and ideal blood flow
  - silicone oil coated tip
  - three-facets grinding
- Maximum comfort thanks to thin 30G (ø 0.30 mm) or 28G (ø 0.36 mm) lancet tip
- Compatible with lancing devices from B. Braun, Abbott, Bayer, LifeScan, Menarini, Owen Mumford, Sanofi Aventis and Ypsomed

Size	Sales unit (pcs/pack)	REF
28G	200	9167536E
30G	200	9167510E
30G	50	9167554A



## Alcohol Pads B. Braun

- Non-woven swabs
- Saturated with 70 % isopropyl alcohol
- For skin cleansing prior to puncture or injection

Size	Sales unit (pcs/pack)	REF
32.5 x 30 mm (folded)	100	9160612
65.0 x 30 mm (unfolded)		

B. Braun Melsungen AG | 34209 Melsungen | Germany  
 Tel. +49 5661 71-33 99 | Fax +49 5661 71-35 50 | [www.bbraun.com](http://www.bbraun.com)