



ASKINA® DRESSIL

FOAM DRESSINGS WITH SILICONE ADHESIVE

B. Braun Wound Care. Right. From the start.

Askina® DresSil | Askina® DresSil Border | Askina® DresSil Sacrum

Foam dressings with silicone adhesive



Gentle and secure adherence

Minimized trauma during dressing changes

Significant absorption capacity

Easy to use

Good conformability on difficult-to-dress areas and skin contours


Repositionable, can be cut to shape

Improves patient comfort

Hypoallergenic

Askina® DresSil is a self adherent foam dressing with soft silicone adhesive on one side and a vapour permeable waterproof film on the other. It combines the absorption capacity of the foam with the soft adhesion of the silicone contact layer.

Structure of Askina® DresSil

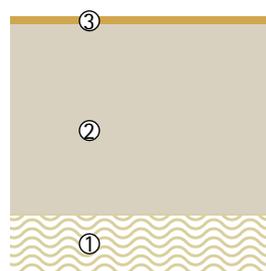

1. Protective polyurethane film

Highly breathable
Bacteria and waterproof barrier

2. Hydrophilic foam layer

Soft and conformable
Excellent absorption capacity

3. Silicone adhesive layer

Safe adhesion to the surrounding skin with atraumatic removal
Silicone layer is perforated to allow exudate to pass through to the foam layer

TIP

- ▶ Askina® DresSil can be used under compression bandages
- ▶ Askina® DresSil may also be used as an aid for the prevention of skin breakdown
- ▶ Askina® DresSil Border allows the patient to take a shower



Indications

- ▶ Pressure ulcers
- ▶ Venous leg ulcers
- ▶ Arterial leg ulcers
- ▶ Diabetic foot ulcers
- ▶ First and second degree burns
- ▶ Donor sites
- ▶ Skin tears
- ▶ Difficult to dress areas

① Wound in epithelializing phase

objective: protection of newly formed tissue and atraumatic dressing removal

② Optimal exudate management

③ Within 2 days, the wound is completely epithelialized

④ Healed wound with very satisfying cosmetic result

Askina® DresSil Border has the same structure as Askina® DresSil, with an additional 1.5 cm large adhesive border, for more security during wear. It is specially adapted for difficult-to-dress or moving areas (knees, elbows, skin folds).

Askina® DresSil Border in use:



Askina® DresSil Border is also available in a shape specifically designed for the sacral area, Askina® DresSil Sacrum.

TIP

- ▶ Ideal for use with Askina® Calgitrol Paste on infected diabetic foot ulcers.



New! Novel shape Askina® DresSil Border

Specifically designed for diabetic foot ulcers, the large silicone adhesive border with circular shaped island of foam is perfectly suited for small difficult wounds. Suitable for covering deep sinuses or cavity wounds.



Askina® DresSil Heel

Anatomically shaped foam dressing with silicone adhesive



Specifically designed for wounds on the heel, Askina® DresSil Heel is a new foam dressing with silicone adhesive. It is easy to position and is self-adherent, so there is no need for secondary fixation.

The foam layer provides excellent absorption capacity and retention properties. Soft and conformable it has a cushioning effect which gives more protection and helps to relieve pressure.

Indications

Moderately exuding chronic and acute wounds located on the heel:

- ▶ Pressure ulcer
- ▶ Diabetic foot ulcer
- ▶ Skin tears
- ▶ Venous and arterial leg ulcer
- ▶ First and second degree burns
- ▶ Donor sites

Soft silicone adhesive layer

- ▶ Atraumatic dressing removal
- ▶ Repositionable, allowing wound inspection
- ▶ Coated all over the foam to allow intimate contact

Foam layer

- ▶ Excellent absorption capacity
- ▶ Soft and conformable
- ▶ Adaptable shape

Large Border

- ▶ Security for improved patient comfort
- ▶ Showerproof

TIP

- ▶ Askina® DresSil Heel can also be used for prevention of pressure ulcers
- ▶ Protects the heel and the malleolous
- ▶ No need for additional fixation



How to position Askina® DresSil Heel

- ① Remove the transparent top liner that is protecting the two "ears".
- ② Position the foot on the dressing.
- ③ Cover the ankles with the "ears" of the dressing. Remove the transparent lower liner.
- ④ Wrap and cover the heel with the lower part of the dressing.

Askina® DresSil	Size overall dressing foam island	Pcs/Pack	Article number
	5 cm x 7 cm	3 10	5295703 5295710
	10 cm x 10 cm	3 10	5291003 5291010
	15 cm x 15 cm	3 10	5291503 5291510
	10 cm x 20 cm	3 10	5291203 5291210
	20 cm x 20 cm	5	5292005
Border	6 cm x 6 cm 3 cm Ø	10	5396610
	7.5 cm x 7.5 cm 4.5 cm x 4.5 cm	3 10	5397503 5397510
	10 cm x 10 cm 7 cm x 7 cm	3 10	5391003 5391010
	10 cm x 20 cm 7cm x 17 cm	10	5391210
	15 cm x 15 cm 12 cm x 12 cm	3 10	5391503 5391510
	15 cm x 20 cm 12 cm x 17 cm	3 10	5395203 5395210
	20 cm x 20 cm 17 cm x 17 cm	5	5392005
Sacrum	16 cm x 17.5 cm 13 cm x 14.5 cm	5	5491605
	21 x 22 cm 18 cm x 19 cm	5	5492105
	21 x 22 cm 18 cm x 19 cm	10	5492110
Heel	22 cm x 21.6 cm 18.9 x 18.5 cm	5	5592205



Manufactured by: B. Braun Hospicare Lt. | Collooney | Co. Sligo | IRELAND
www.bbraun.com

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun.

Non binding document and photographs.