



## IryPump® S

TIME MACHINE – 48 HOURS OF FREEDOM ... OVER AND OVER AGAIN  
Colonic Irrigation

# IryPump® S

Time machine – 48 hours of freedom ... over and over again

Shopping, travelling, sports – or what you have in mind. With IryPump® S, the innovative electrical pump for stoma irrigation, you can enjoy a new quality of life.

Simple and secure use, surprisingly compact and portable everywhere thanks to the high performance storage battery; irrigation with IryPump® S provides you with the independence and freedom you have been waiting for:

- » Self-determined defecation
- » Significant reduction of intestinal gas-forming bacteria (unwanted flatulence)
- » Reduces the stoma treatment to a minimum
- » Up to 48 hours between use during which you can finally do just what you want



## The new generation of mobile electrical irrigation pumps

- Innovative, compact design
- Easy, comfortable operating
- Individual selectable flow control settings
- Easy handling for everybody
- Stable standing – to place and take anywhere
- Adjustable and constant water flow
- Comfortable, easy and safe irrigation through electrical pump
- Adjust the pump to your personal needs



## Bi-material face plate

Extra soft backside »» comfort for body

»» high water tightness

Rigid front side »» easy to place the sleeve on

Multiple belt fixation »» good fixation

»» adjustable to body




## Universal sleeves

Skin friendly adhesive »» 3 options to use:

on face plate, skin or

Softima®/Flexima® Key (60 mm) base plate

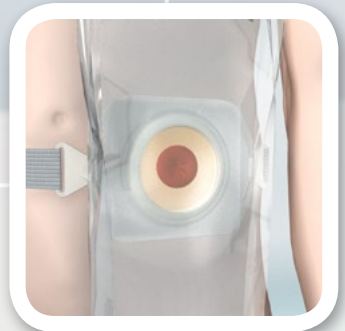

## Cone

Soft material »» high comfort

»» easy to use

»» good handling

Latex and DEHP free

## Mini pouches for your free time

Discrete, comfortable and secure

»» Iryfix®, B. Braun Petite,

Softima®/Flexima® Active Mini



## Prerequisites and contraindications for stoma irrigation

### + General conditions


- The colostomy is located at the end of the descending or the sigmoid colon
- A normal bowel function exists with regular, formed stools
- The patient is familiar with the handling of the normal stoma treatment and is personally able to carry this out
- The patient can irrigate regularly at the same time

### — Contraindications

- Colon prolapse
- Hernia
- Siphon-effect
- Inflammatory bowel disease
- Colon stenosis
- Radiotherapy and chemotherapy
- Poor health conditions
- Cardiovascular disease
- Operation in the abdominal area or the rectum during the past 3 months

### Important recommendation

Before setting out irrigation procedures, a physician/nurse consultancy is compulsory. It is recommended that the first irrigation with IryPump® should be carried out under the guidance of a trained physician/nurse in consideration of potential risks.

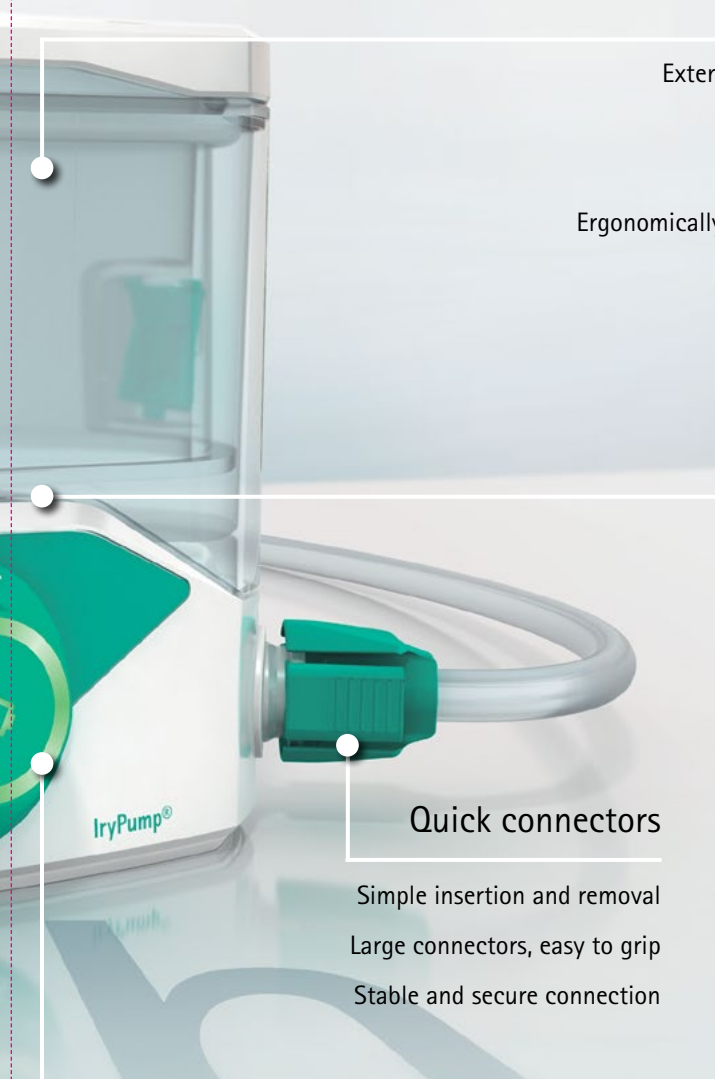

## Water container

- Extendable and removable water container
- Easy to read level indicator
- Easy to fill and clean
- Ergonomically designed, compact and easy to carry



## Temperature indicator

- Temperature control while filling



## Quick connectors

- Simple insertion and removal
- Large connectors, easy to grip
- Stable and secure connection

## Control unit

- Easy to operate
- Ergonomically designed
- Integrated battery indicator
- Individually adjustable water flow



## Further information

Additional information about the product portfolio and its application are available on our website: [www.iryump.com](http://www.iryump.com)



IryPump® S Set	Quantity	REF
Complete starter set, contains	1 set	29110
① IryPump® S (electrical pump, water container and power supply)		
② IryCone®+ (cone and tubing)		
③ IrySleeve® Irrigation Sleeves 60 mm		
④ Iryflex® Faceplate		
⑤ B. Braun StomaCare Belt		
⑥ Iryfix®		
⑦ B. Braun Petite		
⑧ Softima®/Flexima® Active Mini		
Carrying case		
		
Same starter set available without Iryfix® and Travel bag	1 set	29120

IryPump® S Accessories	Quantity	REF
② IryCone®+ (cone and tubing)	1 piece	29220
IryPump® Power supply	1 piece	29030
IryPump® Water container	1 piece	29240
③ IrySleeve® Irrigation Sleeves 60 mm	20 pieces	60620
	30 pieces	60630
	50 pieces	60650
④ Iryflex® Faceplate	1 piece	F05064
⑤ B. Braun StomaCare Belt	1 piece	5028
Travel Bag	1 piece	29230

Stoma caps, Mini pouches	Size	Quantity	REF
⑥ Iryfix® Stoma cap with integrated filter	ø up to 35 mm	15 pieces	F00014
⑦ B. Braun Petite Mini pouch 82 x 71 mm	≈ 15-35 mm	30 pieces	F00011
	35 mm	30 pieces	F00015
⑧ Softima®/Flexima® Active Mini Mini pouch beige (~220 ml)	≈ 15-45 mm	30 pieces	4610215



[www.iryump.com](http://www.iryump.com)

Please check the QR-Code with your smartphone/tablet for more information about the new IryPump® Irrigation System.



Discover the advantages of the new, innovative IryPump® Irrigation system with our new IryPump® App. Get it in the App Store for free now.

Manufactured and distributed by:  
 B. Braun Medical | 204 avenue du Maréchal Juin | 92100 Boulogne-Billancourt | FRANCE  
 Tél. 01 41 10 53 00 | Fax 01 41 10 53 99 | [www.iryump.com](http://www.iryump.com)  
 B. Braun Medical | SAS with a share capital of 31 000 000 € | RCS Nanterre 562050856

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun. Non binding documents and photographs.