
# Cystofix® SG set

## Cystofix® Exchange sets

---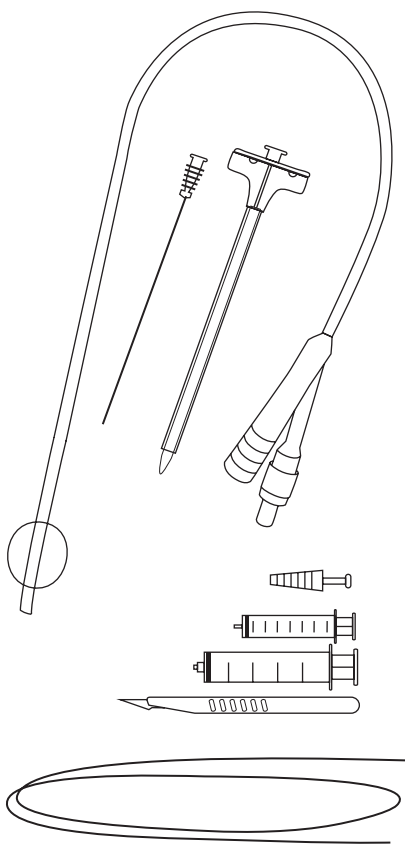

- EN Instructions for use
- DE Gebrauchsanweisung
- ES Instrucciones de uso
- FR Mode d'emploi
- IT Istruzioni per l'uso
- NL Gebruiksaanwijzing
- BG Инструкции за употреба
- CS Návod k použití
- DA Brugsanvisning
- EL Οδηγίες χρήσης
- ET Kasutusjuhend
- FI Käyttöohjeet
- HR Upute za upotrebu
- HU Használati utasítás
- LV Lietošanas instrukcija
- LT Naudojimo instrukcija
- NO Bruksanvisning
- PL Instrukcja obsługi
- PT Instruções de utilização
- RO Instrucțiuni de utilizare
- RS Uputstvo za upotrebu
- RU Инструкция по применению
- SK Návod na použitie
- SL Navodila za uporabo
- SV Bruksanvisning
- TR Kullanım Kılavuzu
- ZH 使用说明

## Puncture set, exchange set and dilation exchange set for suprapubic percutaneous catheterisation.

### Contents

#### Cystofix® SG puncture set

Drainage catheter with balloon, 18G 12cm needle, scalpel, guide wire, dilator and splittable sheath, 2 syringes, stopper.

#### Cystofix® exchange sets / dilation + exchange sets

Drainage catheter with balloon and stopper. Guide wire (both exchange sets/dilation + exchange sets), dilator and splittable sheath (only exchange + dilation sets).

### Indications

Drainage of the bladder after surgery, in case of bladder dysfunction, urinary retention, acute prostatitis, abnormal urethral anatomy, in pelvic trauma, for diagnostic purposes or monitoring of diuresis.

### Contraindications

Carcinoma of the bladder, uncorrected coagulopathy, inappropriate bladder filling (at least 300 ml), skin infection at the designated puncture site, previous lower abdominal surgery, prosthetic devices in lower abdomen (e.g. hernia mesh, vascular bypass).

### Side effects

Despite the special design and material of the catheter, deposition of urinary components and cellular debris may occur that can block the drainage channel, leading to urinary retention. This will be promoted by change of urinary pH, concentration of urine and/ or urinary tract infection. Irritation of the mucosa by the foreign body can cause bladder spasm and urgency.

### Important remarks

- Please take into account current guidelines and recommendations (Harrison SCW et al., BJU International 2010, EAUN Guidelines 2013).
- Suprapubic catheter insertion must be undertaken only by clinicians who have acquired the appropriate skills.
- Suprapubic catheter exchange should be car-

ried out only by adequately trained staff.

- Do not use if the packaging is damaged.
- Do not puncture laterally from the median line of the body: risk of bleeding and injury to intra-abdominal organs !
- Do not insert the guide wire with the stiff tip first !
- Do not inflate the balloon more than indicated on the funnel.

Cystofix® silicone balloon catheters may be used for short and long term suprapubic catheterization.

Inflate the balloon with an adequate fluid (usually sterile water). For long term use of balloon catheters it is advised to fill the balloon with the specified volume by using a sterile mixture of glycerine and water in proportion 1 to 9. This prevents unintentional balloon deflation. Check regularly catheter patency; the physician will decide how and how often this check should be done.

Re-use of single-use devices creates a risk for the patient or user. The device can be contaminated and/or its functional capacity impaired. Contamination and/or limited functionality of the device may lead to injury, sepsis and death of the patient.

**Other indications:** None

**Interactions:** None

### Application

#### First puncture

- Open the Cystofix® SG blister pack and prepare the set components on a sterile area.
- Fill the small syringe with 5ml of sterile water and the large syringe with local anaesthetic.
- Connect the catheter to a urine drainage bag.
- Connect the enclosed needle to the syringe filled with anesthetic.
- In accordance with current guidelines and recommendations, the bladder must be filled properly (at least 300 ml) and should be palpable before puncture. If in doubt or in obese patients use ultrasound for bladder tracking and to en-

sure that the planned puncture track is free from interposing bowel loops.

- Anesthetize the skin at the puncture site when the procedure is not done under general or regional anaesthetic.
- Make a small incision on the median line of the body, 2-3 cm above the symphysis pubis with the enclosed scalpel.
- Puncture vertically the abdominal wall at this position at an angle of 15° to 20° with the prepared needle and syringe while injecting anaesthetic during the passage of the different layers.
- Once the needle has passed the bladder wall urine can be freely drawn through the needle.
- Disconnect the syringe from the needle while maintaining the needle in place.
- Introduce the soft tip of the guide wire via the needle into the bladder.
- Gently withdraw the needle while keeping the guide wire in place.
- Place the dilator and sheath over the guide wire and push them with gentle pressure down into the bladder without moving the guide wire.
- While maintaining the outer sheath in place, withdraw the dilator and guide wire.
- Insert the catheter inside the sheath and push half of its length into the body (the 16 cm mark must be hidden in the track).
- Inflate the balloon with the 5ml of adequate fluid (see above) prepared in the syringe.
- Withdraw the sheath along the catheter track, split it and discard the halves.
- Apply an appropriate dressing to the skin around the catheter.

#### Exchange of catheter

- Open the Cystofix® blister pack and prepare the set components on a sterile area.
- Fill the bladder to approx. 50%.
- If appropriate, insert the guide-wire into the catheter with flexible tip first.
- Deflate the balloon (if appropriate) and withdraw the catheter.
- Insert a new catheter, over the guide wire if appropriate, and inflate the balloon.
- Apply an appropriate dressing to the skin around the catheter.

#### Exchange of catheter with dilation

- Proceed as described above, but after removal of the old catheter dilate the puncture track with a dilator and sheath of appropriate diameter.
- Remove the dilator and the guide wire from the sheath, insert the catheter into the sheath and push half of its length into the body (the 16 cm mark must be hidden in the track).
- Inflate the balloon.
- Withdraw the sheath along the catheter track, split it and discard the halves.
- Apply an appropriate dressing to the skin around the catheter.

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B. Braun Medical SAS