


HOW TO USE

Uro-Tainer® Suby G and/or Solutio R

SIMPLIFIED CATHETER MAINTENANCE


Uro-Tainer® is a sterile, ready to use, disposable system for the maintenance of catheters.

It considerably reduces the formation of crystals, the risk of urinary tract infections and the frequency of catheter replacement. The entire operation is carried out in a closed circuit with no entry or exit of air.

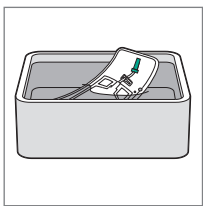


STEP 1

Warm Uro-Tainer® in lukewarm water. Meanwhile disinfect your hands thoroughly. If possible wear disposable gloves.

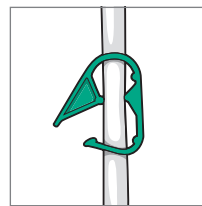


STEP 5

Open the clamp and let a few drops of the Uro-Tainer® solution flow into the catheter to remove any air locks.

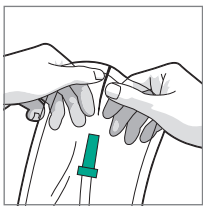


STEP 2

Tear open plastic overwrap and remove the Uro-Tainer®.

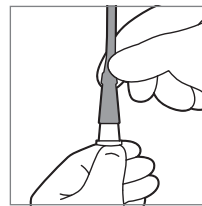

STEP 6

Insert connector of Uro-Tainer® into catheter and let the Uro-Tainer® flow into the catheter as shown in step 7.

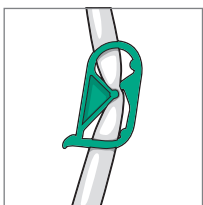


STEP 3

Close the clamp by dipping it.



STEP 7

Allow fluid to drain in by gravity until no more flows in. Do not squeeze the sachet. Leave in position for 5 minutes closing the Uro-Tainer® clip. Do not disconnect the sachet from the catheter.

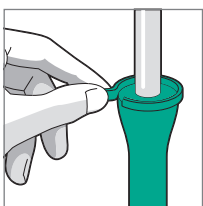

STEP 4

Pull tag to remove cap. Twist and withdraw cap without touching sterile connector.



STEP 8

Open clip and hold bag downwards to allow the used solution to drain back into the Uro-Tainer® sachet. When the bag is full, close the clip and remove from the catheter. Attach the drainage bag and discard the Uro-Tainer® after use. Now disinfect your hands. For single use only.