Prontosan® - Wound Gel

Instructions for use for cleansing, moistening and decolonization of acute and chronic skin wounds, 1st and 2nd degree burns

1. Introduction: Chronic skin wounds are usually coated with slough, necrotic tissue and/or biofilm. These coatings are extremely difficult to remove and may delay normal wound healing. Therefore proper wound cleansing is essential. Because Prontosan® - Wound Gel can be left on the wound bed for many days, the application of this moisturizing gel provides continuous cleansing and debridement between dressing changes. Such a cleansing and debriding property is also indicated in acute wounds since these are generally contaminated with foreign bodies, debris and micro-organisms. It is well known that in acute wounds such a contamination can interfere with the normal wound healing process, cause a delay in healing and give rise to complications like infection.

In acute traumatic wounds where suturing is necessary, the Gel can be applied after surgical intervention

2. Product profile and indications for use:

For cleansing, moisturizing and debridement of

a) acute wounds: - traumatic wounds such as lacerations, abrasions or stab wounds (if suturing is indicated Prontosan® - Wound Gel should be applied after surgical intervention);

- postoperative wounds

b) chronic wounds (especially undermined and difficult to access wounds) including, arterial and venous ulcers, diabetic ulcers, pressure ulcers

c) thermal wounds, (I. and II. degree burns, frostbites)

d) chemical and radiation-induced wounds

For preservative moistening of bandages and wound dressings, such as compresses, gauze, pads, sponges, hydrofibres, alginates, hydrocolloids etc.

For loosening of encrusted bandaging or other encrusted wound dressings during dressing changes

The use of Prontosan® - Wound Gel is painless and also well-tolerated by allergy sufferers. Unpleasant wound odour is rapidly alleviated.

3. General use: Firstly wounds should always be irrigated and cleansed with Prontosan® - Wound Irrigation Solution (see separate product information) so that debris or coatings that can be readily released have already been removed before further treatment is carried out with Prontosan® - Wound Gel.

The wound bed should always be copiously coated with the gel. For this purpose Prontosan® - Wound Gel is applied to the wound or filled into the wound cavities or wound pockets. Dressings, gauzes, compresses or other absorbent materials or wound fillers can also be wetted or moistened with Prontosan® - Wound Gel in order to use these to cover or to fill the wounds before a dressing is applied.

Prontosan® - Wound Gel can stay on the wound until the next change of dressing. Depending on the frequency of dressing changes, varying amounts of Prontosan® - Wound Gel are applied:

- larger quantity: e.g. when the dressing is changed only after several days.

- smaller quantity: when the dressing is changed on the next or the same day.

In this way the surface of the wound is kept continuously moist, which ensures good cleansing and debridement. Coatings are gently released and removed with the next change of dressing. Application should be carried out frequently enough for all coatings and necroses to be readily removed and for optimal wound bed preparation. This is a good precondition for natural healing of the wound

4. Cleansing the wound site: Thorough cleansing of the surrounding skin of the wound with Prontosan® - Wound irrigation solution is recommended to minimise the risk of germs being transmitted into the wound.

5. Tissue tolerability and biocompatibility: Non-irritant; well-tolerated; painless; no inhibition of granulation and epithelialisation; evaluated as dermatologically harmless.

The excellent tissue compatibility of polihexanide and undecylenamidopropyl betaine has been proven over many years of clinical use, even in patients with long-standing chronic wounds.

6. Side effects: In very rare cases there may be a mild burning sensation after application of Prontosan® - Wound Gel but this disappears after a few minutes. Prontosan can cause allergic reactions such as itching (urticaria) and rashes (exanthema). In rare cases (less than 1 out of 10,000), anaphylactic shock has been reported.

7. Contraindications: Prontosan® - Wound Gel should not be used if the patient is known to suffer from allergies or if it is suspected that the patient may be allergic to one of the ingredients. Prontosan® - Wound Gel should not be used on the CNS or the meninges, in the middle or inner ear or in the eyes.

Use on hyaline cartilage and in aseptic joint operations is contraindicated. If however Prontosan® - Wound Gel does come into contact with aseptic cartilage, it should immediately be irrigated with Ringer solution without lactate or physiological saline.

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Prontosan® - Wound Gel should also not be combined with other wound cleansing soaps, ointments, oils or enzymes etc. When such substances are removed from the wound, it should be ensured that these have been thoroughly rinsed off (e.g. with NaCl).

8. Restrictions of use: Pregnancy and lactation period: There are no hints showing any mutagenic or embryotoxic effects of the ingredients. Since there is no systemic resorption of Polihexanide, a pass over of the ingredients to the breast milk is unlikely. Due to the lack of relevant trials and long-term clinical experience with pregnant and breast feeding women, Prontosan® - Wound Gel should only be used after careful medical consultation in this case.

Newborns and infants: Due to insufficient clinical experience, Prontosan® - Wound Gel should only be used selectively in newborns and infants.

9. General safety instructions: Only for external use on skin wounds. Do not use for infusion or injection! Do not swallow. Use only unobjectionable and undamaged bottles. Keep bottles out of direct sunlight.

- Medical device - Keep out of reach of children!

10. Summary / technical information: Prontosan® - Wound Gel: Purified Water, Glycerol, Hydroxyethylcellulose, Betaine Surfactant, 0.1 % Polyaminopropyl Biguanide (Polihexanide).

Appearance and smell: clear, colourless and virtually odourless, aqueous fat-free gel.

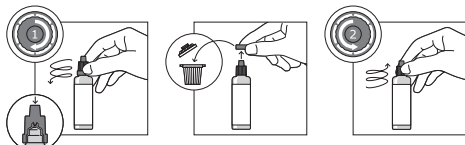
Shelf-life: according to the expiry date; storage at room temperature

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Shelf-life after opening: 8 weeks.

Preservation and shelf-life: Prontosan® - Wound Gel is preserved in such a way that the risk of germs being spread to the direct environment during application is considerably reduced. Therefore, Prontosan® - Wound Gel offers an effective germ barrier between the wound and the patient if used according to the instructions.

Due to preservation, there is a possible shelf-life of 8 weeks after opening the bottle if the bottle is closed immediately after use and the top of the bottle is protected from contamination or direct contact. Bottles that have come into contact with the wound must be thrown away after use.

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400432 1000ml

Prontosan® - Wound Gel 400510 30ml

Prontosan® - Wound irrigation solution

Instructions for use for cleansing, rinsing, moistening and decolonization of acute and chronic skin wounds, 1st and 2nd degree burns

1. Introduction: Chronic skin wounds are usually coated with slough, necrotic tissue and/or biofilm. These coatings are extremely difficult to remove and may delay normal wound healing. Therefore it is important to use a proper cleanser with superior cleansing activity like Prontosan® - Wound Irrigation Solution.

Proper cleansing is also indicated in acute wounds since these are generally contaminated with foreign bodies, debris and micro-organisms. It is well known that in acute wounds such a contamination can interfere with the normal wound healing process, cause a delay in healing and give rise to complications like infection.

Prontosan® - Wound Irrigation Solution disperses well and shows an excellent cleansing activity on the tissue surface, both on biofilms and on difficult to access parts of the body and wound.

2. Product profile and areas for use

For cleansing and irrigation plus moistening of

- acute wounds: - traumatic wounds (e.g. skin lacerations, bites, cuts or crush injuries); - postoperative wounds
- chronic wounds including: pressure sores; venous ulcers; diabetic ulcers
- thermal and chemical wounds: 1st and 2nd degree burns; chemical burns
- post-radiation wounds
- fistulas and abscesses
- entry ports (e.g. urological catheters, PEG-/PEJ-tubes and drainage tubes)
- peristomal skin

For intraoperative cleansing and irrigation of wounds (in explorative operations such as severe injury of the extremities) For instillation during vacuum-assisted closure.

For moistening bandages and wound dressings such as compresses, gauze, pads, sponges, hydro fibre, alginates, hydrocolloids and similar and when changing dressings to dissolve encrusted bandages or other encrusted wound dressings.

3. General use: Firstly the wounds should always be irrigated and cleansed with Prontosan® - Wound Irrigation Solution so that coatings that can be readily released have already been removed before further treatment is carried out with Prontosan® - Wound Gel (see separate product information).

Gauzes or pads soaked in Prontosan® - Wound Irrigation Solution can be used for cleansing as required.

Application should be carried out frequently enough for all coatings and necroses to be readily removed and to achieve an optically clean wound. This is a good precondition for normal wound healing.

4. Cleansing the wound site: Thorough cleansing of a large area around the entire wound site with Prontosan® - Wound Irrigation Solution is recommended to minimise the risk of germs being transmitted into the wound.

For wounds covering a large area and also wounds difficult to access, bathing the affected area of the body with Prontosan® - Wound Irrigation Solution, for at least 15 minutes, is recommended. The solution should be used in an undiluted form.

Note: Patients with chronic wounds are often sensitive to temperature. Prontosan® - Wound Irrigation Solution can be warmed to body temperature immediately prior to use.

5. Loosen wound dressings and encrusted bandages: Bandages are frequently encrusted and also stick to the wound surfaces. If these dressings are removed from the wound surface without being soaked (because of time pressure), it can traumatise the wound bed, which in turn delay wound healing.

In cases where bandaging is difficult to release, soaking the wound dressings with Prontosan® - Wound Irrigation Solution is recommended until they can be gently removed without traumatising the surface of the wound. If large encrustations are present, the whole area of the body with the dressing should be saturated or bathed in Prontosan® - Wound Irrigation Solution until the dressings can be easily removed.

6. Tissue tolerability and biocompatibility: Non-irritant; well-tolerated; painless; no inhibition of granulation and epithelialisation; evaluated as dermatologically harmless.

The excellent tissue compatibility of polyhexanide and undecylenamidopropyl betaine has been proven over many years of clinical use, even in patients with long-standing chronic wounds.

7. Side effects: In very rare cases there may be a mild burning sensation after application of Prontosan® - Wound Irrigation Solution but this disappears after a few minutes. Prontosan can cause allergic reactions such as itching (urticaria) and rashes (exanthema). In rare cases (less than 1 out of 10,000), anaphylactic shock has been reported.

8. Contraindications: Prontosan® - Wound Irrigation Solution should not be used if the patient is known to suffer from allergies or if it is suspected that the patient may be allergic to one of the ingredients.

Prontosan® - Wound Irrigation Solution should not be used on the CNS or the meninges, in the middle or inner ear or in the eyes.

Use on hyaline cartilage and in aseptic joint operations is contraindicated. If however Prontosan® - Wound Irrigation Solution does come into contact with aseptic cartilage, it should immediately be irrigated with Ringer solution without lactate or physiological saline.

Do not use in combination with anionic tensides as these may impair preservation.

Prontosan® - Wound Irrigation Solution should also not be combined with other wound cleansing soaps, ointments, oils or enzymes etc. When such substances are removed from the wound, it should be ensured that these have been thoroughly rinsed off.

9. Restrictions of use: Pregnancy and lactation period: There are no hints showing any mutagenic or embryo-toxic effects of the ingredients. Since there is no systemic resorption of Polyhexanide, a pass over of the ingredients to the breast milk is unlikely. Due to the lack of relevant trials and long-term clinical experience with pregnant and breast feeding women, Prontosan® - Wound Irrigation Solution should only be used after careful medical consultation in this case.

Newborns and infants: Due to insufficient clinical experience, Prontosan® - Wound Irrigation Solution should only be used selectively in newborns and infants.

10. General safety instructions: Only for external use on skin wounds. Do not use for infusion or injection! Do not swallow. Use only unobjectionable and undamaged bottle. Keep bottles out of direct sunlight. Medical device - Keep out of reach of children.

11. Summary / technical information: Prontosan® - Wound Irrigation Solution: Purified Water, Betaine surfactant, 0.1 % Polyaminopropyl Biguanide (Polihexanide).

Appearance and smell: clear, colourless and virtually odourless, aqueous solution.

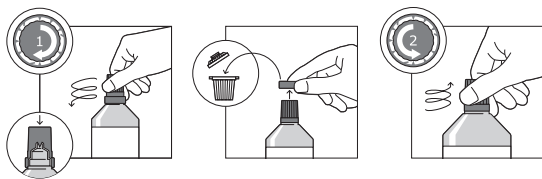
Shelf-life: according to the expiry date; storage at room temperature.

Originality: A - sterile, originality seal.

Shelf-life after opening: 8 weeks.

Ampoules: for single use only!

Preservation and shelf-life: Prontosan® - Wound Irrigation Solution is preserved in such a way that the risk of germs being spread to a wider environment during application is considerably reduced. Therefore, Prontosan® - Wound Irrigation Solution offers an effective germ barrier between the wound and the patient if used according to the instructions. Due to preservation, there is a possible shelf-life of 8 weeks after opening the bottle if the bottle is closed immediately after use and the top of the bottle protected from contamination or direct contact. Bottles that have come into contact with the wound must be thrown away after use. Ampoules: for single use only!

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B.Braun Australia Pty Ltd
B. Braun Medical AG