

# Prontosan<sup>®</sup> Wound Irrigation Solution 40 mL

For cleaning wounds, and for moistening and lubricating absorbent wound dressings

880424 P-7911-3  
REV. 11/15

## Description:

Prontosan Wound Irrigation Solution is a clear, colorless, and nearly odorless waterbased solution used to clean wound surfaces, even those that are difficult to access.

STERILE by aseptic filtration <1 CFU/mL, until the product is first opened.

For single patient use only.

Not made with natural rubber latex or DEHP.

## Ingredients:

Purified Water, 0.1%  
Undecylenamidopropyl Betaine, 0.1%  
Polyaminopropyl Biguanide (Polyhexanide [PHMB]),  
pH may be adjusted with Sodium Hydroxide.

PHMB at a concentration of 0.1% w/w is added to the product as a preservative to inhibit growth of microorganisms within the product.

## Indications:

Prontosan Wound Irrigation Solution is intended to clean wounds and moisten absorbent wound dressings for the management of minor

cuts, abrasions, lacerations and minor burns.

## Adverse Reactions:

In very rare cases there may be a mild burning sensation after application which subsides after a few minutes. Prontosan can cause allergic reactions such as itching and rashes. In rare cases (less than 1 out of 10,000), anaphylactic shock has been reported.

## Warnings:

- Do not use if you have an allergy to one or more of the ingredients.
- Do not use on exposed cartilage.
- Do not use or mix with other cleansers, soaps, lotions, or ointments.
- For external use only on skin wounds.
- Do not use for infusion by IV or injection.
- Do not swallow.
- Do not use in the eye.

## Storage Conditions:

Store at room temperature. Avoid excessive heat. Do not refrigerate or freeze.

## Shelf-life:

Use prior to expiration date.

## Instructions For Use:

Use only as directed. Do not dilute the solution before use.

## Cleaning the Wound Site:

Clean the wound and surrounding area with Prontosan Wound Irrigation Solution before further treatment is carried out.

Bandages or dressings soaked in Prontosan Wound Irrigation Solution can be used for cleaning by applying saturated bandage directly on the wound.

- Rinse wound thoroughly with Prontosan Solution. For thickly encrusted wounds, allow solution to soak in until scab is softened.
- Prontosan Solution can be used to irrigate wound by squeezing directly from container. No rinsing is required afterwards.

- Reapply Prontosan Solution with each dressing change.
- Prontosan Gel can be used to keep the wound moist prior to applying the dressing. Refer to Prontosan Gel Instructions for Use.

## For Use In the Case of Encrusted Bandages or when Changing Dressings Is Problematic:

Saturate the wound dressing with Prontosan Wound Irrigation Solution until the bandages can be gently released without damaging the surface of the wound.

**Directions:** Wash hands before use and avoid contact with the bottle rim or solution.

1. To open container, firmly grasp tab between thumb and forefinger. Twist until tab separates from container.
2. Proceed with use.
3. Do not save and reuse any remaining solution.

# B | BRAUN

Manufactured for:  
B. Braun Medical Inc.  
Bethlehem, PA 18018-3524 USA  
1-800-227-2862  
www.prontosan.bbraunusa.com