DE Prontosan® Wound Spray

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GB Prontosan® Wound Spray

Instructions for use

Indication: For cleansing, irrigation and moistening of
a) superficial acute and superficial chronic wounds (e.g. skin lesions, bites, lacerations, abrasions, crush wounds).
b) superficial burns.
For the cleansing of entry ports such as transurethral or suprapubic urological catheters, PEG-/PEJ-tubes and peristomal skin areas.

Characteristics:

- Supports fast healing by optimal wound cleansing and reduces scarring.
- Reduces odour.
- Prevents wound infection.
- Prevents biofilm formation.
- Prevents MDRO (e.g. MRSA, VRE, ESBL) contamination.
- Moistens wound dressings and dissolves encrusted bandages or wound dressings during dressing changes.
- Reduces pain.
- Painlessly applicable.
- Non-irritant and well-tolerated.
- No inhibition of granulation and epithelialisation.
- Evaluated as dermatologically uncritical.
- Good compatibility with catheter and probe materials.

Use:

CLEANSING AND CARE OF WOUNDS AND BURNS:

Extensively spray Prontosan® Wound Spray onto the entire wound and the wound site. With this, the risk of infection is reduced and optimal healing conditions are generated. Afterwards, cover the wound with a dressing. Clotted or encrusted dressings can be moistened with Prontosan® Wound Spray in order to be easily removed and to avoid pain during removal.

TRANSURETHRAL CATHETERS:

Thoroughly moisten a sterile gauze and cleanse the catheter entry site.

PEG-/PEJ-TUBES AND SUPRAPUBIC CATHETERS:

For cleansing of entry port, apply a sufficient amount of Prontosan® Wound Spray. For encrustations, leave on for at least 1 minute. Dry surrounding skin area by dabbing with a sterile gauze and apply a sterile cover afterwards.

STOMA CARE:

Thoroughly moisten a sufficient number of sterile compresses. Clean peristomal skin areas and dry afterwards. Subsequently, apply a new dressing. Prior to the use of Prontosan® Wound Spray, rinse off the residues of soaps, ointments and oils or other wound cleansing substances thoroughly.

Warnings / Contraindications:

- Only for external use on wounds. Do not use for infusion or injection. Do not swallow.
- In very rare cases there may be a mild burning sensation on the wound surface after application of Prontosan® Wound Spray, but this dissipates after a few minutes.
- Prontosan® products can cause allergic reactions such as itching (urticaria) and rashes (exanthema). In rare cases (less than 1 out of 10,000), anaphylactic shock has been reported.
- Prontosan® Wound Spray must not be used if the patient is known or suspected to suffer from allergies to any of the ingredients.
- Prontosan® Wound Spray may not be used on the CNS, the meninges, in the middle or inner ear, on hyaline cartilage or in the eyes.
- Avoid inhalation of the Prontosan® Wound Spray aerosol.
- Do not use in combination with anionic surfactants as these may interact with polyhexanide.
- Due to insufficient clinical experience with pregnant and breast feeding women, newborns, and infants, Prontosan® Wound Spray should only be used after careful medical evaluation in these patients.
- Do not use damaged bottles. Bottles that have come into contact with the wound must be thrown away after use.
- A physician should be consulted if the wound does not begin to heal within several days, or if swelling, redness or pain develops at the wound site.
- Do not use continuously for longer than 30 days.

Storage information:

- Shelf life 24 months.
- Shelf life after opening 12 months.
- Storage at room temperature.
- Keep bottles out of direct sunlight.
- Medical device – Keep out of reach of children.

Composition / technical information:

Purified Water, Betaine surfactant, 0.1 % Polyaminopropyl Biguanide (Polihexanide).
Appearance and smell: clear colour- and virtually odourless aqueous solution.

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B. Braun Medical AG