Sempach

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# Prontosan® Debridement Pad

## EN Wound debridement pad

**Directions for use**  
Prontosan® Debridement Pad consists of microfibres that debride and an absorbent backing layer. The device is individually packed and sterilised with ethylene oxide. Prontosan® Debridement Pad is intended for single use only and must not be re-sterilised. The device is sterile as long as the pack remains unopened and undamaged.

**Indications**  
Prontosan® Debridement Pad is used for rapid, effective and safe debridement.  
Prontosan® Debridement Pad frees the wound from coatings and dead cell residues (debris) and absorbs excess exudates. Intact tissue is spared.  
Prontosan® Debridement Pad produces good results even with scaly and necrotic coatings, if they are subject to prior autolytic treatment.

**Composition:** polypropylene, polyester  
**Instructions for use**  
1) Remove the old wound dressing.  
2) Perform debridement:  
a) The unmarked side of the pad is moistened with Prontosan® wound irrigation solution or, if not available, with saline solution or distilled water. If local guidelines exist, these must be observed.  
b) Applying gentle pressure, wipe the moistened side over the wound and/or adjacent skin in a circular motion. Following debridement, it is advisable to irrigate the wound thoroughly for a second time with Prontosan® wound irrigation solution and to continue with standard treatment.

**Precautions**  
- Do not use Prontosan® Debridement Pad as a wound dressing.  
- The device must not be used in case of known intolerance or an allergy to one or more of its constituents.  
- Prontosan® Debridement Pad must not be cut to size.

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Title: 880537 - Prontosan Debridement Pad Initiator: Rene ? Bruendler

This document is signed electronically in compliance with the B. Braun electronic signature policies and procedures by following persons:

UserName: Bruendler, Rene (bruerech)  
Title: OM-RA-CH05 Coordinator Labeling Design  
Date: Monday, 26 November 2018, 10:45 W. Europe Daylight Time  
Meaning: Document signed as Author

---

---

UserName: Graber, Monika (steimoch)  
Title: Supply Chain Manager  
Date: Monday, 26 November 2018, 13:24 W. Europe Daylight Time  
Meaning: Approve Document

---

---

UserName: Brechbuehl, Joerg (brecjoch)  
Title: Clinical Research Associate  
Date: Tuesday, 27 November 2018, 08:54 W. Europe Daylight Time  
Meaning: Approve Document

---

---

UserName: Wassmer, Andrea (buchanch)  
Title: Junior Regulatory Affairs Manager  
Date: Wednesday, 28 November 2018, 08:48 W. Europe Daylight Time  
Meaning: Approve Document

---

---

UserName: Gluschke, Michael (glusmich)  
Title: Head of regulatory Affairs  
Date: Thursday, 29 November 2018, 15:54 W. Europe Daylight Time  
Meaning: Final Release of the Document

---

---