Askina® SilNet Plus

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EN INSTRUCTIONS FOR USE

1. PRODUCT DESCRIPTION

Askina® SilNet Plus Silicone Wound Contact Layer dressing is a primary dressing for use when adherence to the wound is a potential problem. The dressing allows exudate to pass into an absorbent secondary dressing. On this skin, the dressing will provide secure adhesion with minimal epidermal stripping or pain on removal.

Askina® SilNet Plus Silicone Wound Contact Layer dressing is a sterile wound dressing, consisting of a knitted polyester fabric coated with silicone adhesive on both sides; this prevents the dressing from sticking to the wound and reduces pain during dressing changes.

Askina® SilNet Plus Silicone Wound Contact Layer dressing can be used in conjunction with Negative Pressure Wound Therapy (NPWT). Not made with natural rubber latex.

2. INDICATIONS

Askina® SilNet Plus Silicone Wound Contact Layer dressing is indicated for use on nil to heavily exuding chronic and acute wounds, as appropriate secondary dressing. The dressing may be used throughout the healing process on skin tears, surgical incisions, second-degree burns, partial thickness skin grafts, pressure ulcers, venous and arterial ulcers.

Askina® SilNet Plus Silicone Wound Contact Layer is a primary dressing. A secondary absorbent dressing is required if exudate is present.

Askina® SilNet Plus Silicone Wound Contact Layer dressings are suitable for use under compression bandaging.

3. CONTRAINDICATIONS

Askina® SilNet Plus Silicone Wound Contact Layer dressings are not indicated for use on individuals with a known sensitivity to silicone.

4. PRECAUTIONS

In the event of clinical infection, consult the relevant healthcare professional.

Do not re-use in whole or in part, as it may compromise sterility and/or the performance of the dressing.

Askina® SilNet Plus Silicone Wound Contact Layer dressings must not be kept away from children and animals.

For use under the guidance of a healthcare professional.

Not for surgical implantation.

When used on burns treated with meshed grafts, avoid placing unnecessary pressure upon the dressing.

When used for the fixation of skin grafts and protection of blisters, the dressing should not be changed before the fifth day post application.

If used in conjunction with NPWT always document the number of cut pieces used in the patient record to ensure that no dressing is left on the skin when the dressing is changed.

If more than one piece of silicone wound contact layer dressing is used, overlap should be minimised to prevent occlusion of holes. No more than two pieces of silicone wound contact layer dressing should be overlapped.

5. INSTRUCTIONS FOR USE

Site Preparation

If necessary, cleanse the wound area according to appropriate woundcare protocols. Dry the surrounding areas of the wound thoroughly before dressing application.

Dressing Application

1. Select a size of Askina® SilNet Plus Silicone Wound Contact Layer dressing making sure the dressing size is slightly larger than the wound. If necessary, the dressing can be cut (using sterile scissors) to the required size, ensuring the Wunddistanzgitter is slightly larger than the wound.

2. The dressing size should extend well beyond the open wound [at least 2cm] in all directions.

3. Remove one release liner from the wound dressing.

4. Centre the dressing on the wound and apply it gently to the

wound site. Press the dressing on the surrounding skin by gentle application of pressure. The second release liner can then be removed and a secondary dressing applied if applicable.

If more than one piece of silicone wound contact layer dressing is required, ensure that the dressings overlap (to avoid secondary dressing adherence to the wound).

6. A secondary dressing can be applied if needed.

Dressing Change and Removal

1. Observe the wound and / or secondary dressing frequently and change either dressing whenever good wound care practice dictates. Askina® SilNet Plus Silicone Wound Contact Layer dressing can remain in situ for up to 14 days, dependent on patient condition and the level of exudate.

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