



BE 1[®] STOMA COVER

THE IDEAL COMBINATION FOR EVERY FACET
OF COLOSTOMATE'S LIFE

Be 1^{®*}: Innovation for left colostomates

Be 1[®] represents a totally new concept: With its compact design, this stoma cover is designed for left colostomates who are looking for an alternative to standard colostomy bags that offers them the accustomed security, more discretion, and control over stoma gases.

- Compact solution: stoma cover with folded colostomy bag inside
- Active gas evacuation: whenever the pressure becomes too high
- Discretion: no crackling sound of a hanging bag and no bowel noises¹
- Safe appliance: leak-proof and with automatic opening if intestinal pressure becomes too high
- Easy and secure fixation: guided Flexima[®] 3S coupling system



Be 1[®] is an addition to our successful Flexima[®] 3S 2-piece system which enables colostomates to switch freely between colostomy bag and Be 1[®] whenever they want.



Flat wafer Flexima[®]
3S Be 1[®]



Be 1[®]
front view



Flexima[®] 3S
closed

**In some documentation as the Instructions for use or the clinical study, Be 1[®] is named capsule or capsule cap.*

Be 1[®]: An answer to patients' major concerns




Helps your patients to




BE UNPERCEIVED AS PATIENT

- Be 1[®] avoids unpleasant noise¹
- Feel comfortable with everything they do

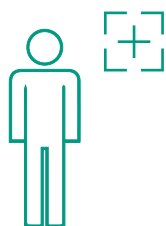

BE CONFIDENT AND ACTIVE

- Be 1[®] improves colostomates' body image
- Helps them to retrieve social life and daily activities
- To have intimate relationships

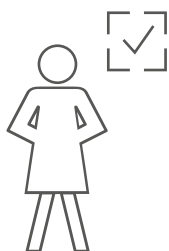

BE THE PERSON WHO DECIDES

- Patient can switch freely between usual colostomy bag and Be 1[®] according to his activity
- Patient can decide where and when to evacuate gases and stools

Offer your patients the ideal combination



- Be 1[®] is a combinable addition to the two-piece Flexima[®] 3S colostomy bag, which works with the same guided coupling system using the same Flexima[®] 3S flat wafer.
- Your patients with a left colostomy now have a real alternative to the standard bags. They can now choose their appliance according to their activity, either the compact stoma cover Be 1[®] or the standard Flexima[®] 3S closed bag.

The right appliance for the right moment



Public event

Intimate life

Leisures

Dancing

Going to cinema

Going to restaurant

Traveling

Sport


Quality of life improved with Be 1[®]

Give your patients the opportunity to leave their fears behind!




Georges, 62 years old, works in a restaurant. For him irrigation is not a satisfying solution to manage noise and gas evacuation.

Now with Be 1[®], I am no longer afraid that customers will notice noises and I can accept working at any time.

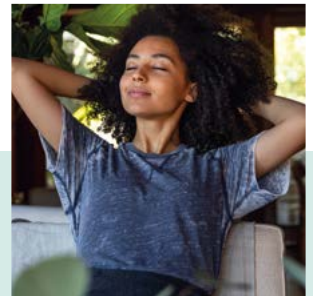

Suzan, 27 years old, has a Crohn's disease and a temporary colostomy.

I could not stand the image of my body. I did not even consider being able to have intimate relationship. Then my nurse told me about Be 1[®]. Now I feel better in my body and I can consider having a life as before.

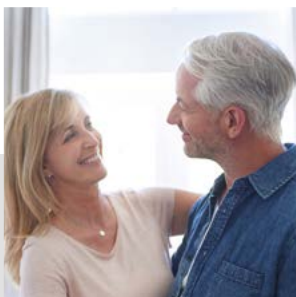

Isabella, 58 years old, is engaged in many different social activities. She loves walking and dancing ballroom.

*For me it was a reborn, I don't see myself as a sick person anymore!
I am free to do all my usual activities, even dancing without fears.*



Helen, 80 years old, was recommended for hydrotherapy by her doctor. However, wearing a stoma pouch on her body blocked her completely and she refused the treatment.

*Thanks to Be 1[®]
I went to a water cure for my arthritis, I feel better and I feel confident now.*



The right way to recommend Be 1[®] to your colostomy patient

BEFORE SURGERY



Allay patient's fears and present all different solutions for people with left colostomy

BAGS FOR LEFT COLOSTOMATES



BE 1[®] STOMA COVER

with the possibility to switch freely to a the Flexima[®] 3S colostomy bag



COLOSTOMY IRRIGATION




SURGERY AFTER SURGERY



Advice appliance that best fits to your patient with temporary or permanent left colostomy

BAGS FOR LEFT COLOSTOMATES AFTER SURGERY

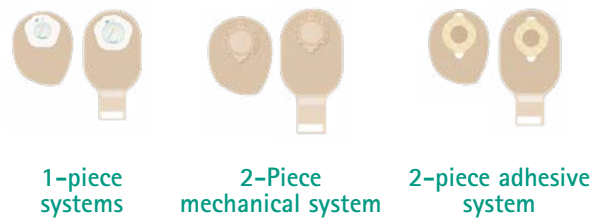

FLEXIMA[®] 3S



If you think that your patient may be interested in trying Be 1[®] later, why not introduce our Flexima[®] 3S two-piece colostomy bags now to facilitate the switch to Be 1[®] during the next visit after a month?

AFTER 1 MONTH



Recommend Be 1® to people looking for an alternative to stoma bags with following profile

- Temporary or permanent stoma
- Left colostomates
- Stoma with a diameter < 40 mm
- Colostomy protruding < 10 mm
- Solid to pasty stool consistency

BE 1® STOMA COVER



THE RIGHT APPLIANCE FOR THE RIGHT MOMENT



AFTER 6 MONTHS



Encourage people to go ahead and try Be 1® to find back into their active life despite having a permanent colostomy.

- To have the choice between different appliances according to their activities
- For more life quality and flexibility
- To retrieve a social life
- To enjoy daily activities

BE 1® STOMA COVER



Or offer colostomy irrigation which has to be regularly performed at a fixed time of the day



Iryflex® Set













IryPump® S Set

How to choose the right appliance size

Stoma diameter	Be 1® Stoma Cover	Wafer
Inferior to 30mm	∅ 55mm	9365xx
Inferior to 40mm	∅ 65mm	9366xx

References

BE 1® STOMA COVER	∅(mm)	Box of	References	Volume ISO (ml)
Box of 10 or 30 	∅ 55	30	9350355	+/- 630 ml
	∅ 55	10	9350155	+/- 630 ml
	∅ 65	30	9350365	+/- 670 ml
	∅ 65	10	9350165	+/- 670 ml
FLEXIMA® 3S BE 1® FLAT WAVER	∅(mm)	Box of	References	
Box of 5 or 10 	 55	5	936517	
	 55	10	936516	
	 65	5	936617	
	 65	10	936616	
FLEXIMA® 3S FLAT WAFER	∅(mm)	Box of	References	
Box of 5 	 55/15-40	5	936515	
	55/35	5	936535	
	55/40	5	936540	
	 65/15-50	5	936615	
FLEXIMA® 3S COLOSTOMY BAG	∅(mm)	Size	References	Volume ISO (ml)
Box of 30 	∅ 55	Mini	● 931255	+/- 230 ml
	∅ 55	Midi	● 931055	+/- 500 ml
	∅ 55	Midi	○ 931155	+/- 500 ml
	∅ 55	Maxi	● 931455	+/- 655 ml
	∅ 55	Maxi	○ 931555	+/- 655 ml
	∅ 65	Midi	● 931065	+/- 465 ml
	∅ 65	Midi	○ 931165	+/- 465 ml
	∅ 65	Maxi	● 931465	+/- 630 ml
	∅ 65	Maxi	○ 931565	+/- 630 ml

 Cut-to-fit | ● Beige | ○ Transparent

Commercialized and manufactured by :
 B. BRAUN MEDICAL SAS | 26 Rue Armengaud | 92210 Saint-Cloud - France
 Tel. +33 1 41 10 53 00 | Fax. +33 1 70 83 45 00 | www.bbraun.fr
 B. BRAUN MEDICAL | SAS with a share capital of 31 000 000 € | RCS Nanterre 562050856
 www.bbraun/be1

The images are used for illustration purposes only and any person portrayed is a model.
 This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun.
 Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun,
 is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun.

For most countries a specific letter is added at the end of the radical reference. All volumes are measured according to ISO 8670-2 standard.
 These medical devices are regulated health care products and on the basis of this regulation are certified CE.
 The products commercialized by B. BRAUN MEDICAL shall exclusively be used in accordance with the package inserts which have been updated and included in the boxes.
 Non binding documents and photographs. For healthcare professional use only.