# Askina® DresSil Border Lite

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## EN INSTRUCTIONS FOR USE

### 1. PRODUCT DESCRIPTION

Askina® DresSil Border Lite is a sterile wound dressing that is soft, thin and discreet and conforms to anatomical contours. The dressing is breathable and manages non to low exuding wounds. The dressing is easy to remove and provides a gentle but secure adhesion to skin with minimal epidermal stripping and pain during removal.

Askina® DresSil Border Lite has a trilaminar structure consisting of a hydrophilic, absorbent, polyurethane central foam island, and a breathable, low friction polyurethane film outer layer that provides a waterproof and bacterial barrier, and together with the silicone adhesive coated perforated wound contact layer, forms the border of the dressing.

### 2. INDICATIONS

Askina® DresSil Border Lite is indicated for use in the management of non to low exuding, chronic and acute wounds throughout the healing process. The dressing is indicated for use on the following wounds:

- Pressure ulcers
  - Diabetic ulcers
  - Leg & foot ulcers
  - Post-operative surgical wounds
  - Trauma wounds (including lacerations, abrasions, skin tears and blisters)
  - Superficial and partial thickness burns
  - Radiation damaged skin
- The dressing can be used on fragile skin and is suitable for use under compression bandaging.

### 3. PRECAUTIONS

- Do not use on individuals with a known sensitivity to polyurethane films, foams or silicone.
- Do not use with oxidising solutions such as hypochlorite or hydrogen peroxide, as these can break down the absorbent polyurethane component of the dressing.
- Not for surgical implantation.
- Do not re-use in whole or in part, as it may compromise sterility and/or the performance of the dressing.
- Askina® DresSil Border Lite must not be ingested and must be kept away from children and animals.
- For use by or under the guidance of a healthcare professional.

### 4. INSTRUCTIONS FOR USE

**Site Preparation**

1. Debride when necessary and irrigate the wound site in accordance with standard protocols.
2. Dry the area surrounding the wound thoroughly before dressing application.

### Dressing application

1. Select a size of Askina® DresSil Border Lite making sure the foam pad is slightly larger than the wound. The foam pad size should extend beyond the open wound by at least 2cm in all directions.
2. Centre the dressing and apply gently over the wound site (white site to the wound).
3. Press the dressing on the surrounding skin by gentle application of pressure.
4. The dressing can be repositioned during application and lifted for observation during wear time if necessary.

### Dressing change and removal

The dressing may remain *in situ* for up to 7 days. Dressing change frequency will depend

on patient condition and level of exudate and it may be necessary to change the dressing more frequently.

2. Change the dressing when it has reached its absorbent capacity or whenever good wound care practice dictates.
3. To remove the dressing, lift one corner of the dressing and slowly peel back until completely removed from the wound bed.
4. Dispose of the used dressing according to local procedures and guidelines.
5. If required, irrigate the wound site with a suitable cleanser prior to application of a new dressing.

### Distributed by: B Braun Hospicare, Collooney, Sligo, Ireland.

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